

FULL-ARCH IMPLANT-RETAINED PROSTHESIS

6 Appointment Workflow

1st Appointment

Doctor takes preliminary impression using custom tray from Mariner.

2nd Appointment

Doctor takes final open tray impression, used for fabrication of the following:

- 1. Master Cast
- 2. Verification Index
- 3. Bite Rim

3rd Appointment

Doctor takes the following:

- 1. Centric Record
- 2. Opposing Cast Impression
- 3. Study Cast of Existing Prosthesis
- 4. Complete Shade & Mould Selection
- 5. Face Bow (If Indicated)



During this appointment, doctor will use the verification index to confirm the accuracy of the master cast. Changes can be made to master cast if verification index reveals any issues. Once information is received by Mariner, we will mount the casts and set-up the teeth.

4th Appointment

Doctor confirms esthetics and bite relationship.

Mariner will then complete the following:

- 1. Make an Index of the Set-Up to the Cast
- 2. Design and Fabricate the Framework
- 3. Treat & Opaque the Framework
- 4. Reset Teeth on the Framework in Wax

5th Appointment

Doctor completes following tasks and then returns prosthesis to Mariner for processing:

- 1. Confirm Passive Fit of Framework via Screw Test
- 2. Confirms Esthetics, Phonetics, and Function of Prosthesis

6th Appointment

Prosthesis is delivered to doctor with final abutment screws for final placement. The screw access holes can be filled with Teflon tape and capped with composite or acrylic resin.

If you have any questions, please contact the Mariner team at 281-440-9222.